EN Smart Digital Thermometer

Eco Temp Intelli IT (MC-280B)

Instruction Manual

All for Healthcare

1. Introduction

Thank you for purchasing the OMRON Digital Thermometer Eco Temp Intelli IT. This thermometer eliminates any worries about broken glass or mercury hazards. Note that your temperature is affected by smoking, eating or drinking. Contact your doctor if you have any questions regarding specific temperatures.

Intended Use

The thermometer you purchased offers a safe, accurate and quick temperature reading. You can measure your temperature either in the mouth (orally) or underarm (axillary).

Intended Operator

This thermometer is intended to be operated by adults who can understand this instruction manual. It is mainly designed for household use.

2. Important Safety Information

Read the Important Safety Information in this instruction manual before using this device. Follow this instruction manual thoroughly for your safety. Keep for future reference.

Warning Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

- Keep the thermometer out of the reach of infants, toddlers and children. DO NOT allow children to operate/ use the thermometer while unattended. Children may injure themselves or other children when attempting to take temperatures without appropriate supervision.
- To help avoid choking, keep the battery, battery cover and screw away from infants, toddlers and children. If any of these are swallowed, consult with your physician immediately.
- NEVER diagnose or treat yourself based on your temperature readings. ALWAYS consult with your physician.
- DO NOT use the thermometer to measure anything other than human body temperature.
- This thermometer is designed for oral and underarm temperature measurements only. DO NOT attempt to take temperature at other sites, such as in the ear or rectum, as it may result in false readings and may lead to injury.
- DO NOT attempt to take temperature measurements when the thermometer is wet, as this may cause inaccurate readings.
- DO NOT use the device with other medical electrical (IME) equipment simultaneously. This may result in incorrect operation of the device and/or cause an inaccurate reading.

Battery Handling and Usage

- DO NOT throw the thermometer into fire.

Caution Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient, or cause damage to the equipment or other property.

- This thermometer does not have water-resistant functionality. DO NOT immerse the thermometer in water.
- DO NOT use the thermometer in places where strong static electricity or electromagnetic fields are present. It may cause an inaccurate measurement result and/or cause the thermometer to operate incorrectly.
- DO NOT allow the thermometer to be shared among individuals without properly disinfecting the device.
- DO NOT disassemble or attempt to repair the thermometer. It may cause inaccurate reading and/or result in incorrect operation of the thermometer.
- DO NOT bite on the thermometer. It may cause breakage and/or injury.
- DO NOT pull or bend the thermometer.
- DO NOT use this thermometer in places with high or low humidity or high or low temperatures. Refer to section 9.
- DO NOT drop the thermometer or storage case or subject them to strong shocks or vibrations.
- During measurement, make sure that no mobile device or any other electrical devices that emit electromagnetic fields is within 30 cm of this thermometer. This may result in incorrect operation of the thermometer and/or cause an inaccurate reading.

Battery Handling and Usage

- DO NOT insert the battery with its polarity incorrectly aligned.
- ONLY use a CR1220 lithium button battery with this thermometer. DO NOT use any other type of battery.
- Remove the battery when the thermometer will not be used for 3 months or more.
- If battery fluid gets in your eyes, immediately rinse with plenty of clean water. Consult with your physician immediately.
- DO NOT replace the battery while the power is on.
- DO NOT use the battery after its expiration date.

3. Measuring Temperature Correctly

Measurement accuracy cannot be guaranteed when the thermometer is incorrectly used to take a temperature measurement.

Oral use (for ages 4 and older):

- The mouth should remain closed up to 5 minutes before measurement.
- Place the thermometer in the mouth under the tongue so that it rests to the left or right of the root of the tongue.
- Use downward tongue pressure to hold the thermometer in place.
- Hold the thermometer to keep it from sliding around in the mouth.

- Open the transfer screen of the app on your smartphone.
- While "°" symbol flashes, press the power button. The thermometer will be ready for transferring your temperature measurement result.
- Note**
 - Your temperature measurement result cannot be transferred after the thermometer is turned off.
 - When the thermometer is ready for transferring, you may hear noise from the thermometer. It is not a malfunction.
 - It is recommended to transfer your temperature measurement result in a quiet place.

Go to next column for the next step

6. Place the thermometer on your smartphone's microphone as the illustration to the right.

The transmission will start automatically and will continue transferring the measurement reading to the app for the next 2 minutes until the thermometer is turned off.

- Note**
 - For further details regarding transferring your temperature readings to your smartphone, refer to the transfer screen of the app.
 - Check within the app to make sure your temperature is successfully transferred before turning off the thermometer.
 - If you feel any discomfort from the thermometer's sound, turn off the thermometer immediately after the data transfer.

7. Press the power button to turn off the thermometer.

Clean and disinfect the thermometer before returning it to the storage case. Refer to sub-section 7.2.

- Note**
 - Your temperature measurement result will automatically be stored in the thermometer's memory. The thermometer can save only the last measured temperature.
 - The thermometer will automatically turn off after:
 - 2 minutes from the start of the measurement transfer.
 - 30 minutes after a measurement was initiated but the measurement result was not transferred.

5. Replacing Battery

Battery: CR1220 lithium button battery (commercially available) Use the battery within the recommended period indicated by the "A" manufacturer. Replace the battery if " " and "A1" appear when the thermometer is turned on.

Do: Upon waking it is recommended to take a measurement before starting your morning routine because this can increase your body temperature. It is recommended to measure directly after waking up or rest 30 minutes before taking a measurement.

- Taking a temperature measurement immediately after exercising, bathing, eating or drinking.
- Do: Wait at least 30 minutes before taking the temperature.
- Taking a temperature measurement after moving around after awakening.
- Do: Upon waking it is recommended to take a measurement before starting your morning routine because this can increase your body temperature. It is recommended to measure directly after waking up or rest 30 minutes before taking a measurement.
- Taking a measurement repeatedly.
- Do: Turn the power off and wait at least 30 seconds, and then take a temperature again.
- Taking a measurement while your underarm is sweating.
- Do: Wipe off any underarm perspiration before taking the temperature.
- Taking a measurement after being under the blankets for a long time.
- Do: Cool down the heat, then take a temperature.
- The sensor tip is in contact with clothing.
- Do: Place the sensor tip in the center of the underarm and ensure that it only comes into contact with the skin, not clothing during measurement.

4. Using Thermometer

To transfer your measurements/readings to your smartphone, first download and install the "OMRON connect" app and follow the instructions within the app.

- Press the power button.
 - All symbols appear on the display as an internal test function, the indicator lamp lights and the buzzer beeps.
 - The battery symbol appears every time the power is turned on. It is not a sign of the battery replacement.
 - The most recent temperature measurement result with "m" is displayed.
 - When "°C (°F)" flashes, the thermometer is ready for use.
 - The display will show "L" (for the actual temperature if it is stored in warm ambient environment above 32.0 °C).
- 2. Apply the thermometer to the measurement site.** Refer to section 3 to apply the thermometer correctly.
- 3. The buzzer will emit (beep-beep-beep) sound 3 times when the temperature becomes stable.**

- "°" symbol and indicator lamp flash.
- DO NOT insert the battery with its polarity incorrectly aligned.
- ONLY use a CR1220 lithium button battery with this thermometer. DO NOT use any other type of battery.
- Remove the battery when the thermometer will not be used for 3 months or more.
- If battery fluid gets in your eyes, immediately rinse with plenty of clean water. Consult with your physician immediately.
- DO NOT replace the battery while the power is on.
- DO NOT use the battery after its expiration date.

3. Measuring Temperature Correctly

Measurement accuracy cannot be guaranteed when the thermometer is incorrectly used to take a temperature measurement.

Oral use (for ages 4 and older):

- The mouth should remain closed up to 5 minutes before measurement.
- Place the thermometer in the mouth under the tongue so that it rests to the left or right of the root of the tongue.
- Use downward tongue pressure to hold the thermometer in place.
- Hold the thermometer to keep it from sliding around in the mouth.

- Open the transfer screen of the app on your smartphone.
- While "°" symbol flashes, press the power button. The thermometer will be ready for transferring your temperature measurement result.
- Note**
 - Your temperature measurement result cannot be transferred after the thermometer is turned off.
 - When the thermometer is ready for transferring, you may hear noise from the thermometer. It is not a malfunction.
 - It is recommended to transfer your temperature measurement result in a quiet place.

Display/Problem	Possible Cause	Solution
"AH-" appears.	The thermometer is/was stored in a location where the temperature is/was 40 °C (104 °F) or more.	Take a temperature measurement immediately after leaving the thermometer in a location where the temperature is between 10 °C and 40 °C (50 and 104 °F) for at least 1 minute.
"AL-" appears.	The thermometer is/was stored in a location where the temperature is/was lower than 10 °C (50 °F).	Take a temperature measurement immediately after leaving the thermometer in a location where the temperature is between 10 °C and 40 °C (50 and 104 °F) for at least 1 minute.
When the thermometer is turned on "L" appears.	The temperature of the sensor tip is lower than 32 °C (89.6 °F).	This is not a malfunction. Keep taking a temperature measurement.
When the thermometer is turned on "H" appears.	The temperature of the sensor tip is higher than 42 °C (107.6 °F).	Cool the sensor tip with a damp towel and then take a temperature measurement.
When the thermometer is turned on a random reading between 32.0 to 42.0 °C (89.6 to 107.6 °F) appears.	The sensor tip is warm and the display shows the actual ambient temperature, if the ambient temperature is higher than 32 °C (89.6 °F).	
" " flashes any time.	Battery is low.	Replacing the battery with new one is recommended. Refer to section 5.
Nothing appears on " " and "A1" appear when the display when the power button is pressed.	Battery polarity is not properly aligned.	Check the battery installation for proper placement. Refer to section 5.
Difference in temperature between 2 consecutive measurements or between different thermometers.	Battery is depleted.	Immediately replace new battery. Refer to section 5.
The temperature readings are inconsistent.	The thermometer is being placed at a different measurement site for each measurement.	Apply the thermometer correctly. Refer to section 3.
Transferring the temperature is not successfully completed.	The thermometer is too far from your smartphone's microphone. The environment during the transfer of the temperature is too noisy.	Bring the thermometer closer to your smartphone's microphone. Transfer your temperature in a quieter place.

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Product category	Clinical Electronic Thermometers
Product description	Smart Digital Thermometer
Model (code)	Eco Temp Intelli IT (MC-280B)
Sensing unit	Thermistor
Measurement accuracy	±0.1 °C (32.0 to 42.0 °C) / ±0.2 °F (89.6 to 107.6 °F) (when measured at a standard room temperature of 23 °C (73.4 °F) in a test tank held at constant temperature)
Temperature display	3 digits (°C), or 4 digits (°F) display in 0.1 degree increments
Measurement position	Oral, underarm
Measurement time	Underarm: approximately 10 minutes; Oral: approximately 5 minutes
Power supply	3 V DC, CR1220 lithium button battery
Durable period (service life)	5 years
Battery life	Approx. 1.5 years or more (1 time per day)
Measurement method	Actual temperature measurement (non-predictive)
Operating mode	Continuous operation
Applied part	Type BF (sensor tip and probe)
Protection against electric shock	Internally powered ME equipment
Measurement range	32.0 to 42.0 °C (89.6 to 107.6 °F)
Memory	1 Memory
Operating conditions	+10 to +40 °C (+50 to +104 °F), 10 to 85% RH (non-condensing), 700 - 1060 hPa
Storage/Transport conditions (between uses)	-20 to +60 °C (-4 to +140 °F), 10 to 90% RH (non-condensing)
Weight	Approx. 14 g (with battery installed)
Dimensions	Approx. 20.1 mm (w) x 36.8 mm (l) x 12.8 mm (d)
Package contents	Main unit, CR1220 lithium button battery, storage case, instruction manual

- The specifications may be changed without prior notice.
- The sensor tip of this thermometer contains nickel.
- This thermometer does not contain any servicable part. OMRON recommends that the accuracy is verified every two years by an OMRON representative.
- Operation mode classification is in accordance with IEC 60601-1.
- This device fulfills the provisions of the European Standard EN12470-3:2000+A1:2009, Clinical thermometers - Part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device.
- Please report to the manufacturer and the competent authority of the Member State in which you are established about any serious incident that has occurred in relation to this device.
- This OMRON product is produced under the strict quality system of OMRON Healthcare Co., Ltd., Japan.

Display/Problem	Possible Cause	Solution
Any of 0 - 4 appears with "Er".	The thermometer may have a malfunction.	Contact your local OMRON representative.

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